
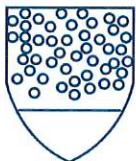
# Fabric Description

TYPE 5

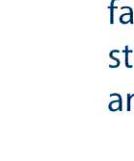TYPE 6

ISO 9001: 2015  
ISO 13485:2016  
OHSAS 18001: 2007  
ISO 37001:2016  
ISO 26000-2010-SA8000  
ISO 14001:2015

As a national key responsive supply entity, our company's Procurement of fabric and trims is in accordance with national standards.

The entire process and raw materials are up the highest standards.

The fabrics, seal tapes and zippers are in accordance with national standards. As a national key responsive supply entity we assists the government in developing various new protective suits.

The protective suit is made of PP spunbond non-woven fabric laminated with PE film. The raw material is high in strength, soft in touch, skin-friendly, comfortable to wear, and strong in antistatic performance.

It has good anti-infiltration performance, high filtering efficiency, surface moisture resistance, strength and air permeability, which allows it to effectively prevent dust and water.

RAW MATERIAL SPECIFICATIONS				
<b>Composition</b>		NW + FILM		
<b>Film</b>		Breathable		
<b>Color</b>		White		
TEST RESULTS				
SPECIFICATION	UNIT	VALUE	TOLERANCE	TEST METHOD
<b>Nonwoven Weight</b>	g/m <sup>2</sup>	20	± % 5	EDANA 40.3-90
<b>PE Film Weight</b>	g/m <sup>2</sup>	35		
<b>Hotmelt Weight</b>	g/m <sup>2</sup>	2		
<b>Total Weight</b>	g/m <sup>2</sup>	57		
<b>Tensile Strength MD</b> * at peak	N / 1 inch	40	± %20	EDANA 20.2.89
<b>Tensile Strength CD</b> * at peak	N / 1 inch	15	± %20	EDANA 20.2.89
<b>Elongation MD</b> * at peak	%	80	± %20	EDANA 20.2.89
<b>Elongation CD</b> * at peak	%	70	± %20	EDANA 20.2.89
<b>Water Vapour Transfer Rate(WVTR)</b>	38°C-90% RH 24 h (Macon)	2000	± %20	WSP 70.5
<b>Hydro Head</b>	mmSS	2400	± %10	DIN-EN 20811
<b>Lam.Peeling Strength</b> * Nonprinted area	N / 1 inch	1,3	± %20	
<b>Storage Condition</b> ; Keep in cool,dry place away from open flame and sunshine				
<b>Guaranteed storage time:</b> 1 year after manufacturing				

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TEK KULLANIMLIK

**SAFETY INSTRUCTION**  
All protective coveralls must be checked before use against defects and faults which may cause negativity such as tears, holes, rips, and cuts. If it is detected and pity, it should definitely not be worn.

**ATTENTION!**  
Playing with a dog is dangerous and can cause suffocation. Please keep away from children and infants.

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### DIRECTIONS FOR USE

#### Main Use Areas

5/6 model Protective coveralls of type KEEP protect workers against hazardous material particles (type 5) as well as against chemical liquid splashes (type 6).

#### Restrictions Of Use

Attention should be paid when removing the coveralls exposed to liquid or dry and the contact with hazardous materials should be avoided. It is not designed for extreme hazardous conditions and toxic chemical substances. The coverall's seams do not constitute a barrier against infectious substances and are not protective. The user is the sole responsible and shall take the final decision regarding the use ability for any application of the 5/6 model Protective coverall of type KEEP. In case that the coverall is used in combination with other personal protective products, it is necessary to tie the cuffs to the gloves, the wristbands to the boots and the respiratory device to the cap. The user is the sole decision maker regarding the conformity of the combination.

It is absolutely necessary for the person wearing the coverall to be properly grounded. The resistance between the person and the ground should be less than 10  $\Omega$  ohm. These conditions can easily be created by wearing the appropriate shoes. If the characteristics of the clothing are not considered, wearing the chemical protective clothing can create a heat stress. The product life cycle and the effect of the working environment on the usage comfort should be planned in advance. Appropriate underwear measures will help to reduce the heat stress.

#### How To Wear It?

How to wear the coverall, be sure that the current area is clean. Open properly the packaging without damaging the product. Open the zipper of the product and then, Pass firstly your feet through the coverall not crumple and break too much the coverall when doing this operation. all, pe our body 15 completely covered up by the coverall, close the zipper after wearing the coverall's cap one 2 mead. REMOVE the adhesive tape on the zipper side and paste it to the coverall by paying attention. Do not fold it. In this way the permeability of the zipper will be avoided.

#### Storage / Final Use

It is recommended to keep it in cardboard or cardboard box, away from sunlight, between 15 - 25 °. If stored on suitable conditions, it is recommended to use it within 3 years after the production date.

#### Destruction/ Recycling

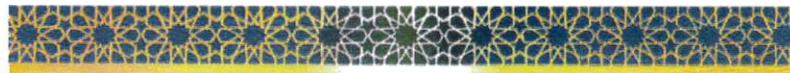
The uncontaminated products can be treated as general waste or disposed of in accordance with the rules laid down by law.

Physical Performance and Resistance Of The 5/6 Model Protective Coverall of Type KEEP	Coverall of Type KEEP
EN ISO 530 Wear resistance	2/6
EN ISO 7854-4: Friction Cracking	6/6
EN ISO 9073-4: Tear strength	2/6
EN ISO 1394-1: Os EN 1149-5:2008	1/6
EN 863 Puncture strength	2/6
Surface resistance internal and external aoe	EN 1149-5:2008-Pass

Resistance Of The 5/6 Model Protected Coverall of Type KEEP	
Tip 5: Particle Test	Pass
Tip 6: Low level Spray Test	Pass
Radioactive particles	EN 1073-2 Class 1
Sewing Strength	Class 3

	Do Not Wash
	Do Not Dry
	Do Not Iron
	Do Not Bleach
	Do Not Use

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PRODUCTION, SALES AND EXPORT OF MEDICAL FACE MASK AND COVERALL

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MANUFACTURING, SALES AND EXPORT OF READY EXTERNAL WEAR

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The Certificate Verification String can be checked by scanning the QR Code on the document or by entering the "TÜRKAK BDS No." and the <https://tbs.turkak.org.tr> document verification system.

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Thia certificate is effective if it is compiled with the certification rules of YBM and  
periodic surveillance audits are completed successfully.

Please consult following numbers for detailed information.

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#### We herewith declare;

The undersigned Company declares under its sole  
responsibility that the item of product specified below  
satisfies the essential requirements of the Medical Device  
Safety Directive. The following products are applicable. The item  
of product identified below has been subject to internal  
manufacturing check assessment by Yekenevi Hizmet Gıyim  
San. ve Tic. A.Ş.

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#### Applicable EU Directives

MEDICAL DEVICE SAFETY DIRECTIVE 93/42/EC

#### Applicable Standards

TS EN ISO 14693: 2012 Medical Devices:  
Application of Risk Management to Medical  
Devices General Rules  
EN 14683: 2019 Medical Face Mask Standard  
EN 1041: 2009 Information Provided by the  
Manufacturer with Medical Devices

## AT UYGUNLUK BEYANI / EC DECLARATION OF CONFORMITY

### AT UYGUNLUK BEYANI / EC DECLARATION OF CONFORMITY

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#### We herewith declare;

The undersigned Company declares under its sole  
responsibility that the item of product specified below  
satisfies the essential requirements of the Personal  
Protective Equipment(PPE) Directive 2016/425/EC which  
are apply to it. The internal production of the product  
described below is Yekenevi Hizmet Gıyim San. ve Tic. Inc.  
controlled by.

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20

#### Product Name

Disposable Hooded Coverall with Lamination and without  
Lamination

#### Type-Models

YPP-01 YPP-02

#### Last Two Digit Year of CE marking affixing

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2016/425 AB KİŞİSEL KORUYUCU EKİPMANLARI